


# Adhesive Bonding Steps – Self Etch Technique

Multilink® Automix

Recommended Cement for All Types of Restorative Materials and All Classes of Restorations

1. Prepare Restoration surface according to manufacturer's recommendations

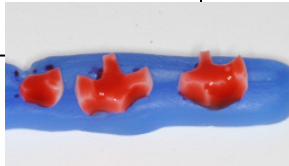

2. Prime etched ceramic surface with Silane (or Prime metal and zirconia surfaces with Metal/Zirconia Primer)



5. Load Cement into restoration, seat and light cure



4. Apply Self-Etching Primer/Adhesive to preparation



3. Clean Preparation surfaces with water and air dry, leaving tooth structure moist

IPS Empress® / IPS e.max®

Internal Surface Etched with Hydrofluoric Acid

Silane Coupling Agent - Monobond-S™

Multilink® Automix Cement

Self-Etching Bonding Agent - Multilink® Primer A/B

Preparation Surfaces Cleaned with Water

Tooth